www.minos97.com

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<http://www.minos97.com>

MINOS CRONOS

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INSTRUCTION AND
MAINTENANCE MANUAL

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Thank you for purchasing a MINOS wheelchair. We are a young company, but we have over 30 years' experience in this sector. The wheelchair which you have just purchased is the result of that experience.

You will not be disappointed with the quality, design and durability of this product.

Read this instruction and maintenance manual carefully. If you have any questions, please consult the expert who advised you to purchase the wheelchair.

Do not forget to have the guarantee stamped, and send it to us within 15 days of the purchase date.

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SAFETY WARNINGS

Components

All components and parts of this chair are original: they must not be replaced by other standard parts available on the market. The use of any component not original to Minos 97 will render the warranty on the chair null and void forthwith. Consult your distributor before changing any component.

Arm and Footrests

Ensure that the arm and footrests are properly secured. Never lift the chair by the arm or footrests, as this could result in injury to the user or damage to the structure of the chair. Never stand on the footrests. Ensure that the footrests are far enough off the ground to avoid impacts.

Wheel Brakes

These brakes are not intended to slow the chair down. Do not apply the brakes while the chair is moving.

Ramps and Obstacles

Practice rolling over an obstacle and travelling up a slope with a technician, occupational therapist or other professional health-care expert before attempting these moves on your own. Have someone watch over you until you acquire the necessary skill. Do not attempt to travel up or down slopes of more than 1 in 10 without assistance. Roll over obstacles with both front wheels (or both rear wheels if you prefer to back over them) at once. Ensure that the foot plates do not bump against the ground on uneven terrain.

Tyres

Ensure that the tyres maintain the recommended pressure on the sides. Tyres must be properly inflated for the chair to manoeuvre correctly, and for the parking brake to grip correctly.

Getting In and Out

First engage the parking brake. Do not use the footrests for support. Normally, the arm rests should be folded back. Move in and out from the side or front, as recommended by a technical advisor, occupational therapist or physiotherapist. Do not move in or out of the chair without a helper on hand until you become skilled at doing so.

Safety in Vehicles

This wheelchair is not to be used as a seat in a vehicle of any kind. Users should be moved into the seat of the vehicle itself.

Folding the Wheelchair

Mind your fingers when folding and unfolding the wheelchair: pull up along the seat upholstery to fold the chair, and push on the supporting tubes under the seat to unfold it.

Leaning Forwards or Back

Do not suddenly lean forwards or back, as this may cause the chair to overturn. If you need to lean back, keep your back on the backrest at all times, and ensure that the front wheels are pointing forwards. If you need to lean forward, centre the front wheels, engage the parking brake and do not lean further than the arm rest.

Outdoor Use

Ensure that your manoeuvres can be seen at all times by pedestrians and especially by drivers if you need to cross the road. Cross only at permitted points.

Maximum User Weight

This wheelchair can be used by persons weighing up to 110 kg.

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Footrest:

70° detachable and swing-aside footrests.

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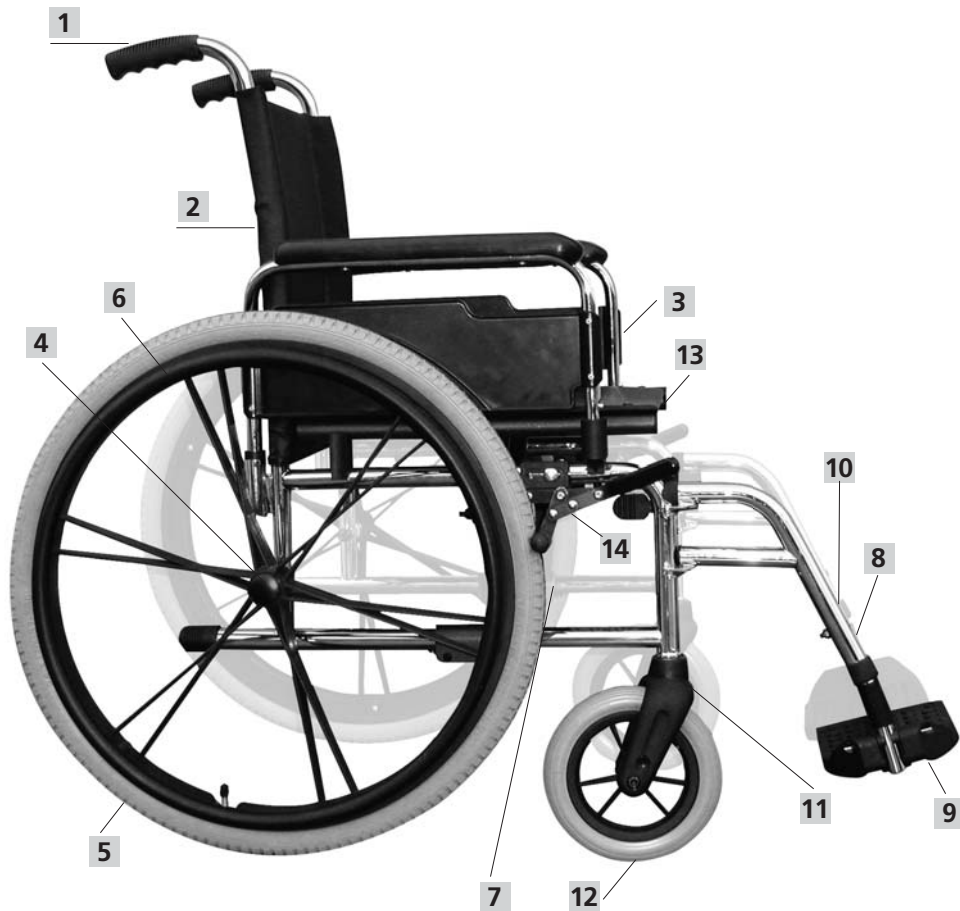
600 and 315 mm, with solid or pneumatic tyres, and normal or sport spokes.

Frenos / Brakes:

Tipo empujar para accionar / Push to engage" type.

CARACTERISTICAS CHARACTERISTICS

- 1** Asidero de empuje / Hand grip.
- 2** Respaldo / Backrest.
- 3** Apoyabrazos desmontable / detachable arm rests
- 4** Eje fijo de la rueda.
- 5** Rueda trasera / Rear wheel.
- 6** Aro de propulsión / Aluminium push rim.
- 7** Cruceta plegable / Folding cross-brace.
- 8** Reposapiés abatibles y desmontables.
Swing aside and detachable footrests.
- 9** Paleta del reposapiés / Foot plate.
- 10** Cinta sujetapiernas / Heel loop.
- 11** Horquilla de la rueda delantera / Front wheel fork.
- 12** Rueda delantera / Front wheel.
- 13** Asiento acolchado / Padded seat with pocket..
- 14** Freno de estacionamiento / Parking brake.

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Footrests

The footrests is detachable. To remove it, press the lever and turn it outwards, then pull it up. To replace it, reverse the process.

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Arm Rest

The arm rest is detachable. To remove it, press the button and pull it back.

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Brakes

To engage the brake, push the lever forwards. To release it, pull the lever back. We recommend engaging the brake before getting into or out of the wheelchair.

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Folding the chair

Set the foot plates of the footrest upright and pull up the seat upholstery until the chair is folded. Never exert force on the arm rests or footrests to fold the chair.

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Opening the chair

Push down on the tubes supporting the seat upholstery, taking care not to pinch your fingers between the tube and the chassis.

Ensure that the seat tubes are supported on their housings.

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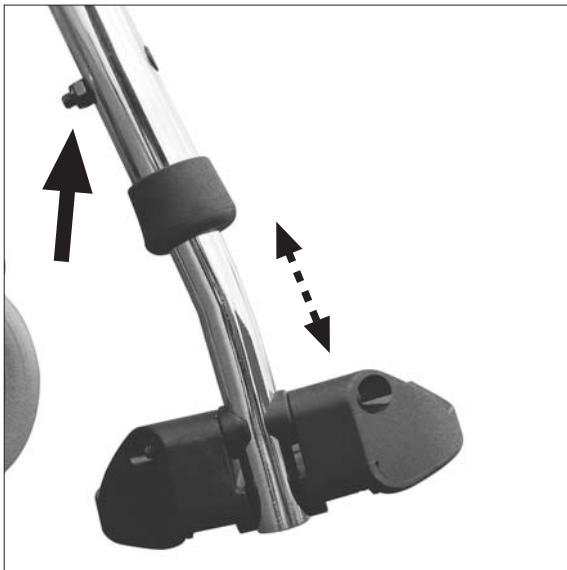
Footrests

To adjust footrest height, release the screw with the spanner and pull the foot plate to the desired height. Then re-tighten the screw.

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Brakes

Release the screws and slide the brake along the tube to the required position.

Re-tighten the screws and ensure that the pad exerts braking force when the lever is pushed.

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Chairs should be serviced by a MINOS dealer once a year, or more frequently in the case of highly active users.

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