MINOS PLENA

MANUAL DE USUARIO
USER'S MANUAL
BEDIENUNGSANLEITUNG

MINOS

ÍNDICE

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MINOS

INDEX

Introduction	16
Important information before using the wheelchair	16
Minos Plena Wheelchair parts	18
Getting in and out of the chair	18
Armrest	19
Safety and operation	19
Folding and unfolding for the car transport	21
Controls and maintenance	22
Repairs and spares	22
Recycling and packaging	21
Minos Plena Wheelchair models	22
Accessories and options	25
Dimensions and general information	28

Introduction

THANK YOU for choosing a MINOS wheelchair. Our company is young but we have broad experience in the sector.

The wheelchair you have just bought is the product of our experience. We are sure that the quality, durability and design of the chair will not disappoint you.

Please read this manual for the use and maintenance of the chair carefully.

If you have any questions, please ask the professional who recommended the purchase of the wheelchair.

Standard

Minos wheelchairs comply with the Medical Devices Directive (1998), which requires certification under the ISO 7176-8 standard.

MINOS

Pol. Industrial Sangroniz
Iberre Kalea 7
48150 Sondika
Vizcaya – Spain
Tel.: + 34 94 453 32 30
Fax: + 34 94 453 30 01
E-mail: pedidos@minos97.com
www.minos.es

DISTRIBUTOR'S STAMP

Lifetime Guarantee

Minos Irati wheelchairs have a lifelong guarantee for the sideframe and crossbar.

IMPORTANT INFORMATION BEFORE USING THE WHEELCHAIR

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How to unpackage your Minos wheelchair

Minos wheelchairs are packaged and protected in their own cardboard box.

To unpackage the chair, pull upwards on the top cover or use a set of pliers to remove the staples.



Taking the chair out of the box

Once the box is open and the chair is out, you will see that the wheelchair is in folded position. On one of the handles, there is a bag containing the instruction manual you are reading.

Serial number

Each chair comes with a sticker containing the serial number on the packaging box and the sideframe.

The serial number identifies the date of sale, batch and establishment to which the chair was sold.

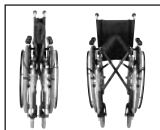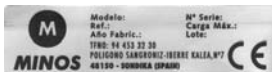Unfolding the chair

Push the tubes that support the seat upholstery outwards, taking care not to trap your fingers between the tube and the frame. Make sure that the seat tubes rest on the supports.

The function of the push rod is to keep the sideframe parallel when folding and unfolding the chair. It also keeps the chair in folded position.

Fitting the footplates

Fit each footplate perpendicular to the sideframe in the hole shown in the photo. Once fitted, turn it 90° until it is fastened in position. To remove the footplates, press the lever, turn each footplate 90° outwards and pull them out.

Tyre pressure

Make sure that the tyres have the pressure recommended on the tyre walls. It is marked on the tyre itself. The recommended pressure is approximately 2.5 kg.

Check that the brakes work correctly.

applied by pushing the lever forwards. To release the brake, pull the lever backwards. The brake spring will keep it separated from the wheel.

We recommend you apply the brake when getting in or out of the wheelchair.

IMPORTANT: The brakes are not to be used to slow down the chair. Do not apply the brake while the chair is moving.

WHEELCHAIR PARTS


- 1 Push handle.
- 2 Backrest.
- 3 Armrest release mechanism for folding, removing and adjusting the height of the armrest.
- 4 Height-adjustable padded armrest.
- 5 Quick-release wheel axle.
- 6 Rear wheel.
- 7 Aluminium handrim.
- 8 Folding crossbar.
- 9 Folding, removable footrests.
- 10 Footplate.
- 11 Heel strap.
- 12 Front wheel fork.
- 13 Front wheel.
- 14 Padded seat with pocket.
- 15 Parking brake.

GETTING IN AND OUT OF THE CHAIR


First of all apply the parking brake. Do not use the footplates for support. Fold the armrest backwards and get in or out of the chair sideways or from the front, as recommended by the technician, occupational therapist or physiotherapist. Do not get in or out of the chair without the supervision of an attendant until you learn to do it easily.

The WC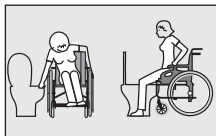


Side approach
Front approach

The bathtub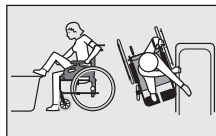


Front approach
Side approach

Transfer to a chair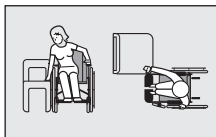

Side approach
Front approach

Transfer from the bed to the wheelchair

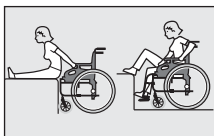Front approach

ARMREST

Folding or removing the armrest

To remove the armrest, press the locking mechanism and pull the armrest backwards. To get in or out of the chair, leave the armrest in this position or remove it completely by simply turning it to release the pin and then pulling it out.

To release the pin, the tab on the shaft must coincide with the groove in the hole.

Adjusting the height

To adjust the height, you need an Allen key and a spanner. Use the tools to unscrew the locking bolt and pull on the armrest until it is at the desired height. Fasten the bolt using the Allen key.

SAFETY AND OPERATION

Parts

All the wheelchair parts and elements are originals and must not be replaced by other standard parts available on the market. The use of any part that is not an original Minos part renders the chair guarantee null and void. Please consult your distributor before changing a part.

Armrest and footrest

Make sure that the armrest and footrest are securely fastened in position. Do not lift the chair by the armrest or footrest under any circumstances, since this could injure the user or damage the structure of the chair. Never stand up on the footrest. Make sure that the footrest is at the correct height from the floor so that it does not come into contact with it.

Pushing the chair yourself

For models with the 600 mm rear wheel.

Use the handrim fitted to the rear wheels.


Slight gradient

Keep your back straight and use the handrim to move forward.

Gentle hill

Bend your back forwards slightly and use the handrim to move forward.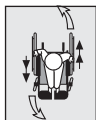


Turning round

Use both handrims, turning them in opposite directions.

Sloping flat (side position)

Keep the side of the body slightly angled towards the higher part of the slope.

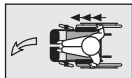Turning

Hold the handrim on the side towards which you want to turn still and use the other handrim to turn.

Wheel brakes

The brakes are not to be used to slow down the chair. Do not apply the brake while the chair is moving.

Centre of gravity

The chair has been designed with the centre of gravity in front of the axis of rotation of the rear wheel. This makes it easier to manoeuvre the chair and prevents it from tipping over.

Anti-tip wheels (optional)

The function of this safety device is to prevent the wheelchair from tipping over backwards. We recommend the use of anti-tip wheels at all times.

Leaning forwards or backwards


These movements could tip the chair over. Consequently, we recommend you make no sharp movements under any circumstances. If you need to lean backwards, keep your back against the back of the chair at all times and make sure the front wheels are pointing forwards. If you need to lean forwards,

centre the front wheels, apply the parking brake and do not lean beyond the armrest.


Falls from wheelchair

If the chair tips over, position yourself in front of it and use your upper members to get in the correct position in the chair. Then ask another person to lift up the chair.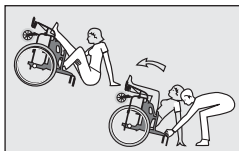


Ramps and obstacles

Before trying to overcome an obstacle or go up a slope, make sure you have practised the manoeuvre with a technician, occupational therapist or other health-care professional. Do not try it on your own the first time. Have an attendant watch your manoeuvres until you acquire the necessary skill. Do not try to go up or down gradients of more than 10 per cent without help. Go over obstacles with both wheels at the same time (front wheels or rear wheels if you prefer to address the obstacle going backwards). Make sure that the footplates do not hit the floor when going up or down steps.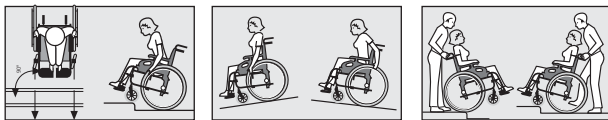

Folding the wheelchair

Take care not to trap your fingers when folding and unfolding the chair. Pull upwards along the length of the seat upholstery to fold it and push the tubes on which the seat rests to unfold it.

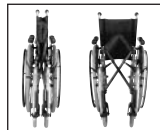Tyres

Make sure that the tyres have the pressure recommended on the tyre wall. Correct tyre pressure is essential for the manoeuvrability of the chair. The correct pressure also affects the hold of the parking brake.

Street use

Make sure at all times that your manoeuvres can be seen by passers-by, especially by vehicles if you have to cross the road. Cross the road at the appropriate place.

FOLDING AND UNFOLDING THE CHAIR FOR CAR TRANSPORT

To transport the wheelchair in the car, choose one of the following two options:

- Place the folded chair in the area between the rear seats and the backrests of the front seats.

- Place the folded chair in the car boot. If you prefer, you can remove the rear wheels with the quick-release axle and the footplates (see ACCESSORIES AND OPTIONS for the corresponding procedure).

CONTROLS AND MAINTENANCE

GENERAL	INITIAL	WEEKLY	MONTHLY	REGULARLY
• The chair can be folded and unfolded easily	■			■
• The chair travels correctly in a straight line	■	■		
BRAKES				
• They do not interfere with the tyres during movement	■		■	
• They can be applied easily	■			■
• They fulfil their parking brake function	■	■		
• They have no play	■		■	
CROSSBAR				
• Check for alignment	■			■
• The four wheels are on the ground	■			■
ARMRESTS				
• They are fitted correctly and can be folded and removed easily	■			■
REAR WHEELS				
• There is no excessive lateral movement when they are turned with the chair empty	■		■	
• Check the tension of the spokes	■	■		
• Check the state of repair of the bearings	■	■		
FRONT WHEELS				
• Check that they are correctly fastened to the fork	■			■
• Check the state of repair of the bearings	■	■		
TREADS				
• Solid: check that there are no cracks	■		■	
• Tyres: check the pressure and state of repair of the tread	■	■		
CLEANING				
• Clean the upholstery on the seat and backrest using soap and water	■			■

REPAIRS AND SPARES

For repairs and spares, visit the establishment where you bought the chair and they will advise you on the best option for your requirements.

RECYCLING AND PACKAGING

The cardboard in which the wheelchair is packaged is 100% recyclable. Please dispose of it in a rubbish container especially for cardboard.

MINOS PLENA

Self-propelled aluminium wheelchair.

Adjust the brake according to the type of tyre, solid or pneumatic. Loosen the two clamp bolts, adjust the section for the brake shoe to work on the tyre and tighten the bolts.

The two positions of the front fork are for fitting 8-, 6- and 5-inch wheels in either of the two positions.

At the top of the fork there are two screws with off-centre hexagonal faces on the fork side. If the screws are loosened and the position of the off-centre hexagon is altered, a different angle of inclination of the fork is achieved. By changing the height of the back wheel, the fork can be positioned perpendicular to the ground.

The front wheel can be changed using an Allen key. Unscrew and remove the bolt and the wheel. Then go through the same steps in reverse. Do not overtighten the bolt so as not to damage the plastic.

The rear frame has series of holes 15 mm separated for varying the height of the seat. To change the position, unscrew the nut and bolt and change the position of the bearing to the new hole.


Converting an Minos Plena in a Minos Plena with Drum Brake

Remove the rear wheel, remove the wheel axle, fit into the wheel axle the drum brake support and fix the drum brake kit. Fit the rear wheel on the drum. At the top of the handle, fit the grip extension and mount the brake handle on the front. Adjust the cable and the sleeve.

How to apply and release the brake

The chair has two brake handles: one on each backrest handle. To apply the brake, squeeze the handles gently until the brake slows down the chair. To release the brake, the handle has one release mechanism that unlocks the brake. When the cable that runs from the handle to the brake regulator is slack, the travel of the brake handle will increase. The cable should be sufficiently tight (not too tight, since that would brake the chair) so as to obtain the correct braking pressure.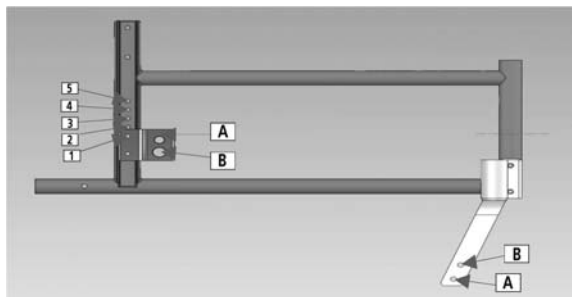

Front Wheel "	Pin position	Pos Plato	Axle plate position				
			1	2	3	4	5
8	A	A	T 465 mm D 500 mm 4,5°	T 450 mm D 500 mm 7°	T 435 mm D 500 mm 9°		
		B	T 490 mm D 500 mm 1°	T 475 mm D 500 mm 3,5°	T 460 mm D 500 mm 5,5°	T 445 mm D 500 mm 7,5°	
	B	A	T 465 mm D 487 mm 3°	T 450 mm D 487 mm 5°	T 435 mm D 487 mm 7°		
		B	T 490 mm D 487 mm 0°	T 475 mm D 487 mm 1,5°	T 460 mm D 487 mm 3,5°	T 445 mm D 487 mm 8°	T 430 mm D 487 mm 8°
6	A	A		T 450 mm D 475 mm 3,5°	T 435 mm D 475 mm 5,5°	T 420 mm D 475 mm 7,5°	
		B		T 475 mm D 475 mm 0°	T 460 mm D 475 mm 2°	T 445 mm D 475 mm 4°	T 430 mm D 475 mm 6°
	B	A	T 465 mm D 462 mm 0°	T 450 mm D 462 mm 1,5°	T 435 mm D 462 mm 3,5°	T 420 mm D 462 mm 5,5°	T 405 mm D 462 mm 7,5°
		B		T 475 mm D 462 mm -1,5°	T 460 mm D 462 mm 0°	T 445 mm D 462 mm 2°	T 430 mm D 462 mm 4°
5	A	A	T 465 mm D 462 mm 0°	T 450 mm D 462 mm 1,5°	T 435 mm D 462 mm 3,5°	T 420 mm D 462 mm 5,5°	T 405 mm D 462 mm 7,5°
		B		T 475 mm D 462 mm -1,5°	T 460 mm D 462 mm 0°	T 445 mm D 462 mm 2°	T 435 mm D 462 mm 3,5°
	B	A		T 450 mm D 450 mm 0°	T 435 mm D 450 mm 2°	T 420 mm D 450 mm 4°	T 405 mm D 450 mm 6°
		B				T 445 mm D 450 mm 0,5°	T 430 mm D 450 mm 2,5°

ACCESSORIES AND OPTIONS

Quick-release axle

Press the quick-release button and insert the axle fully into the hole until it is correctly in position. To remove the wheel, press the quick-release button and pull the wheel outwards.

To adjust the axle, fasten the lower end of the axle with a spanner and use another spanner to adjust the self-locking nut to the desired position.

Anti-tip wheels

These wheels are optional, but they are recommended to prevent the chair from tipping over backwards. To fit them, insert the anti-tip tube into the rear of the frame tube. The distance between the wheel and the ground is adjusted using a clamp system. To remove the anti-tip wheels, remove the clamp.

Stepping pad

The lower part of the rear of the chair has a stopper. To fit the tread, remove the stopper and insert the tread in the tread axle. To fasten the position of the tread once the holes in the frame and in the tread axle are in line, attach these parts together using a

nut and bolt.

The tread can also be fitted using a clamp, with no need for tools.

Small wheels for lifts

At the bottom of the rear of the frame, there is a hole. Insert the lift wheel fork axle in the hole and fasten the bolt.

Replacing fixed upholstery with adjustable upholstery

Remove the side armrests. Unscrew the two bolts at the top of the backrest rear and remove the standard upholstery. Remove the bolts that fasten the seat upholstery and replace it with the adjustable seat.

Replace the bolts, position the crossbar on the supports and tighten the hook and loop fasteners without deforming the crossbar seat

position. Then fit the upholstery using the hook and loop fasteners. Tighten the backrest in the chair position and make sure that the tubes rest on the tube cradle supports so as not to deform the position of the chair. The upholstery should never be over-tightened so as not to affect the chair mechanism.

Commode seat

Remove the bolts that fasten the seat upholstery. Fit the bolt in the commode seat on the front right-hand side of the crossbar. The commode pan has a hole on the front right bridge that fits on to the bolt and holds it in place.

Height-adjustable elevating footrest

To raise the footrest, pull it upwards to the desired height. The footrest automatically locks in position at different heights. To lower the footrest, press the lever on the upper end and push down.

The calf support tilts and is adjusted with the handle on the side.

To adjust the height of the footrest, release the side handle and adjust to the desired height.

Stump support

It has two bolts and four nuts at the bottom of the platform so that the support position can be varied as required.

Both the height and angle can be adjusted.

Reclining backrest with gas piston

This makes it easier to recline the backrest when the person is sitting in the chair. By pressing the two handles, the weight of the person's back helps find the desired position. To raise it, press the two handles again and the power of the piston helps raise the person.

If the gas piston does not work, tighten or loosen the cable regulator located on the top of the handle.

Maximum inclination: 40°. The use of anti-tip wheels is recommended.

Manual reclining backrest

To recline the backrest, pull the release mechanisms until it comes out of position and move it to the desired angle.

Maximum inclination: 60°. The use of anti-tip wheels is recommended.

Long armrest

Remove the side dust guard by unscrewing the bolt and fit the new model (long). Fit at the desired height.

Height-adjustable handles

Unscrew the handles on the top of the backrest, adjust to the desired height and fasten the handles back in position.

IMPORTANT: Do not loosen the handles more than two turns, since the interior nut will come off and unadjust the system.

Safety belt

The safety belt is adjusted using a hook and loop fastener and a clip with flanges at the rear of the backrest handle.

Headrest extension

Insert the extension between the backrest upholstery and the backrest handles until it comes into contact with the backrest upholstery.

Reconditioning kit. This contains the backrest and seat, 2 front wheels and 2 height-adjustable armrests.

DIMENSIONS AND GENERAL INFORMATION

Dimensions	Minos Plena
Seat width	36, 40, 44, 48, 51 cm.
Seat height	50 cm.
Depth	42 cm.
Backrest height	47 cm.
Armrest height	Adjustable from 20-27 cm.
Total width	51, 55, 59, 63, 66 cm.
Total height	94 cm.
Total length	89 cm. 65 cm.*
Width in folded position	31 cm.

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